

USER GUIDANCE

MX™ MESH NEBULISER

MX86200GR



NOTICE

- 1 Please add the liquid to the medication cup before turning on the device.
- 2 Please do not use any health products containing oils in the medication cup.
- 3 Please follow your doctor's advice on drug selection, drug usage and treatment time.
- 4 Please do not use distilled water, purified water, or oil substances for nebulisation.
- 5 Please clean and maintain the nebuliser device after each time it is used.
- 6 Please keep the device, medication cup, and accessories dry after each time they are used.
- 7 Please do not touch the mesh with sharp or hard objects to avoid damage.
- 8 Please charge the device for at least 30 minutes prior to using for the first time.



OPERATING STEPS



To open, rotate the medication cap lid anti-clockwise, following the direction of the arrows on the lid.



Add medication as prescribed by your doctor, into the medication cup. Please pay careful attention not to exceed the MAX level indicated on the cup.



Replace the medication cap lid and turn clockwise to tighten.




Connect the mask or mouthpiece to the spray nozzle on the nebuliser.



TO NEBULISE

1. To start nebulising, press the ON/OFF button once.
2. To stop nebulising, press the ON/OFF button

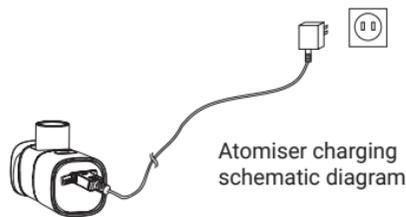

Atomiser charging schematic diagram

Charge the nebuliser device prior to usage, utilising the USB cable provided. To ensure there are no air pockets in the medication cup, gently shake or tap the device to dislodge any air pockets.

Warning: When the medication cup has enough liquid, and the blue light flickers, press the ON/OFF button once to continue nebulising.

Note: Before charging, please make sure the socket connected is normally charged and in good working condition.

Note: Please do not attempt to charge this device with any other charging method other than the USB cable provided.

CLEANING GUIDANCE

PRELIMINARY CLEANING




1 Rotate the mask or mouthpiece clockwise to remove from the nebuliser.



2 Open the medication cup lid and clean any medication residue remaining in the cup.



3 Add an appropriate amount of warm water (below 40°C). Please do not use distilled or purified water.

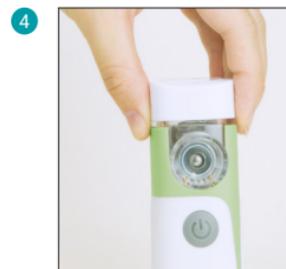

4 Tighten the medication cup lid by rotating clockwise.



5 Please dry the device with clean medical gauze after nebulising. Do not touch the central area of the mesh in case of damage.

Notice: The diagram is provided as a guideline only. The device colour and may differ from the diagram. The information applies to the mechanics of the product, and not to aesthetic detail.

CLEANING GUIDANCE

REGULAR CLEANING



- 1 Hold down the ON/OFF button and pull the medication cup off the device.
- 2 Clean the inside of the medication cup.
- 3 Clean the outside of the nebuliser, including the spray nozzle.
- 4 Clean the inside of the spray nozzle. Please wash the outside and the inside of the medication cup and spray nozzle with purified water (below 40°C). Please dry the device with clean medical gauze. Do not touch the central area of the mesh in case of damage. Please put the device and accessories back in their original place and store as instructed in the manual.

TIP: Nebulising with 3 drops purified water and 1 drop white vinegar will achieve a better cleaning effect.